



In addition to the Blue Sky Bio treatment plan, Implant Concierge also requires the raw DICOM files, as well as a diagnostic cast (a digital impression or a stone model). More information can be found at the bottom of the page.


- 1 In your Acteon software, left click the CBCT tile to highlight it in blue, then right click and select “Export Case”.

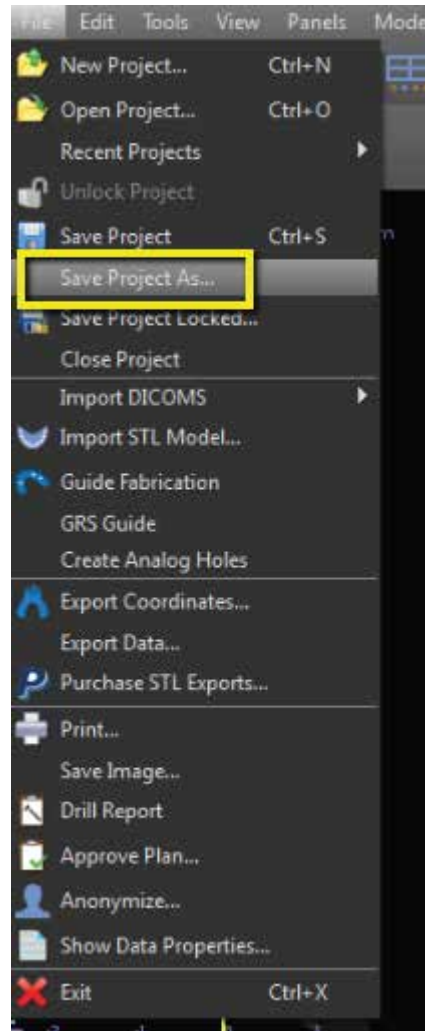


2 Select the Desktop and change the file name to “PatientName_Treatment_Plan.bsb”. Click “OK”

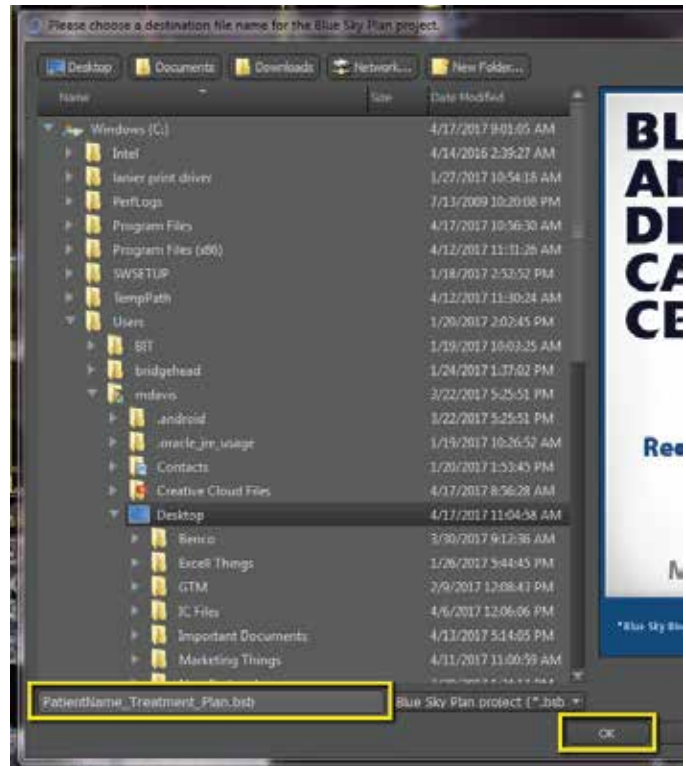


3 After the treatment plan saves, go to the Desktop, right click on the treatment plan, go to “Send to”, and select “Compressed (zipped) folder”. A folder will be made with either a zipper on it, or a blue “Z”.

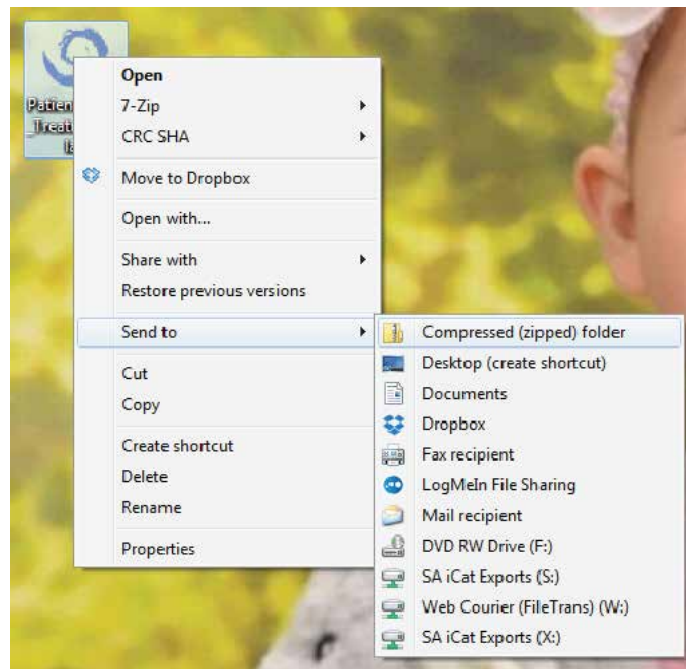

4 To upload the compressed folder, login to Implant Concierge. Click on the patient's name to open the case. Click on the red "Upload DICOM" button.

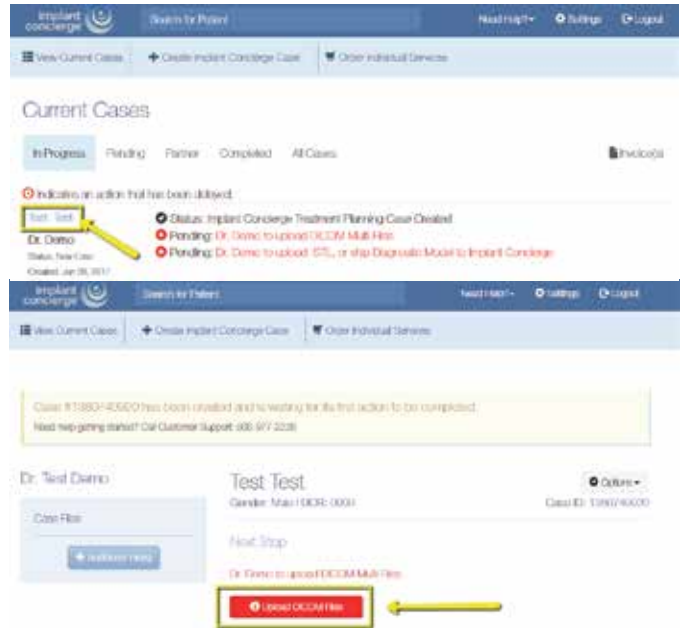

5 Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.

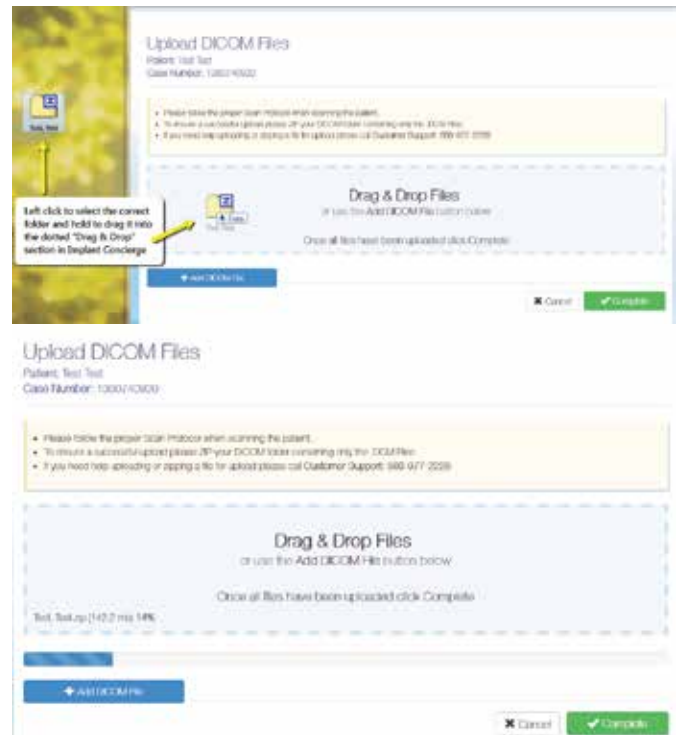

6 Once the file reaches 100%, click the green “Complete” button.

