


1 Open the CD/USB and click “Start”.

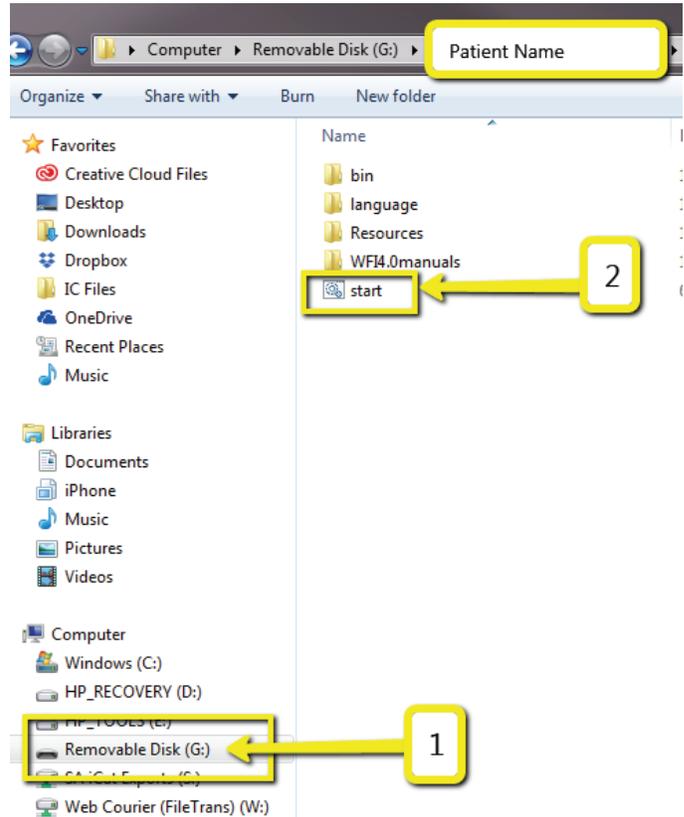

2 Once the CBCT loads in the AIS3DApp, click the folder icon under “DICOM Series”. Click “Zip DICOM series”.




3 A pop-up will appear for the save location. Select “Desktop”. Type the patients name into the “File Name”. Click “Save”.



4 In the upper-right-hand corner, you will see a loading box. There will not be an alert when it has finished exporting.



5 To upload the compressed folder, login to Implant Concierge. Click on the patient's name to open the case. Click on the red "Upload DICOM" button.

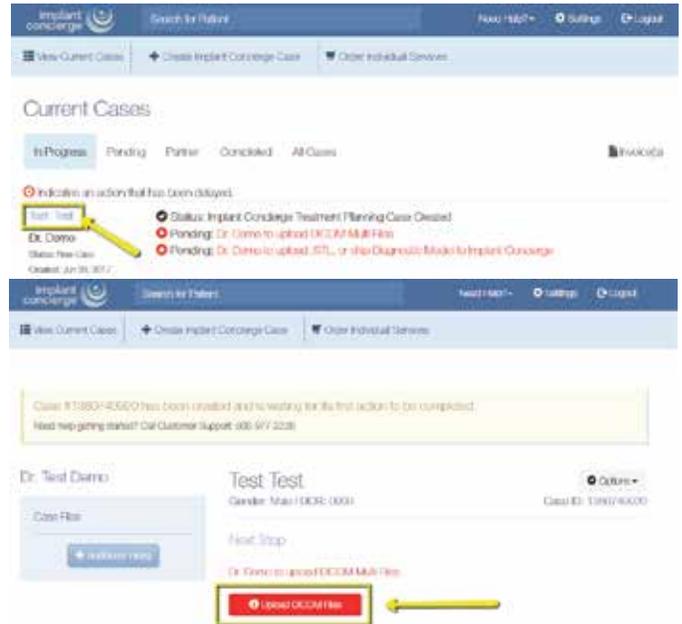

6 Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.



7 Once the file reaches 100%, click the green “Complete” button.

