

**1** Login to Implant Concierge. Click on “Create Implant Concierge Case”



**2** For Implant Concierge to create your treatment plan, select the blue “Create a Case” button.



### Auto Approval Workflow for Single Implant Cases


Choose this option to save time and money by skipping the V.P. meeting. Approve or edit the treatment plan using diagnostic images, clinical measurements and a patient specific video.

Important: Auto Approval is only for single implant cases at this time.

Click Create Case to move on to our case creation page.



### Auto-Approval

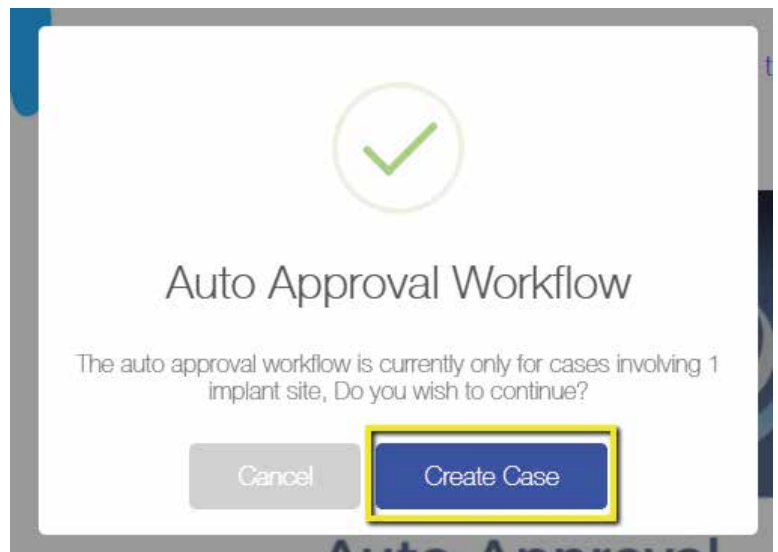


**3** Fill in the blanks with the patient's information and which implant system the Doctor is planning to use.

Then click the green "Next" button.

**4** Fill out the tooth chart on the left (Current Dentate State) to reflect the patient's dentition at the time of the scan. Click on a tooth once to designate it as "missing". Click on the tooth a second time to show a dotted line around the tooth, meaning it will be extracted at the time of surgery. Clicking the tooth a third time places the tooth back on the chart.

For the tooth chart on the right side (Ideal Restorative Plan), this is where the Doctor would ideally like to have the implants placed. Click on the tooth once, to have a virtual implant placed there.

Step 1: Provide Current Dentate State

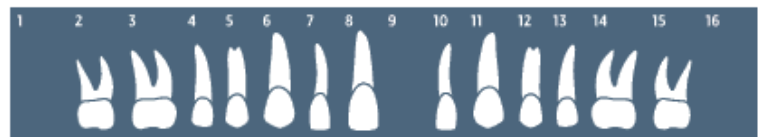

Step 2: Select Ideal Restorative Plan



## 5 Diagnostic Casts/Digital Impressions and Wax ups

This is an optional service. If you are working on a complex case (3 or more teeth consecutively missing) or ordering a provisional, we will need restorative information. You can either choose to send us casts with and without the partial, provide your own wax-up, or choose to have Implant Concierge create a virtual wax-up for you. If you would like Implant Concierge to create a virtual wax-up, we need a cast of the arch of interest, a cast of the opposing arch, and a bite registration. Virtual wax-up pricing will vary depending on the number of teeth requested.

### Diagnostic Model or Digital Impression

**Maxilla:** A diagnostic wax-up is **OPTIONAL** for this arch.

- I elect not to use a wax-up for this case - \$0
- I will provide a diagnostic wax-up or pickup impression - \$0
- Implant Concierge will create a virtual wax-up - \$40

## 6 Provisional Restorations

This is an optional service. If you would like to order a provisional, please select "Yes". Select the Vita tooth shade and occlusion from the drop down box.

### Provisional Restorations

- Yes, I will order Provisionals for this Patient**  
You will customize your provisionals on the next page.

**Select Shade of Provisionals:**

**Occlusion:**

Provisionals require an additional 5-7 business days after your surgical guide has been authorized.

## 7 CAD/CAM Surgical Guide

Please click on "Yes." Keep in mind that this is just an order form! Nothing is charged until the case is completed!

### Surgical Guide

1 implants \$225

- Yes, I will order a surgical guide for this arch

## 8

### CBCT Scan Acquisition

Please specify if one arch was scanned, or if both arches were scanned.



### CBCT Scan Acquisition

- I will furnish my own DICOM files.

#### CBCT Arch Preference

- Scan Maxilla Arch Only
- Scan Both Arches

## 9

### Proposed Restorative Plan

This is where you can make any restorative notes that you would like our team to be aware of when working on this case.



### Proposed Restorative Plan

- Screw-Retained
- Cement-Retained

Provide any notes about the proposed restorative plan

Please add any notes about the proposed restorative plan

## 10

### Radiologist Interpretation

This is an optional service. If you would like a Board-Certified Radiologist to interpret your CBCT scan, then click on "Yes" and select which entity you would like to provide your reading. Please provide your area of interest or instructions to the radiologist in the provided text box.



### Radiologist Interpretation

- Yes, I will order a Radiologist Interpretation

1. Specify area of interest

Please enter any comments about the interpretation


2. Select a Radiologist

<input checked="" type="radio"/>	UT School of Dentistry at Houston 24 - 48 Hour Turnaround	\$ 75
<input type="radio"/>	U.T. Health Science Center 5 - 6 Business Day Turnaround	\$ 75
<input type="radio"/>	Texas A&M University College of Dentistry 24 - 48 Hour Turnaround	\$ 75
<input type="radio"/>	Dr. Michael Morchat (Implant Concierge On-Site Radiologist) 24 - 48 Hour Turnaround	\$ 75
<input type="radio"/>	Dental and Maxillofacial Radiology Omaha 3 - 5 Business Day Turnaround	\$ 112

**11** Once you've reviewed the services and charges you have selected for this case, click on "Create" to start your case!

**12** You are then prompted to authorize that the case is placed into the queue for work up. \*This does not mean that your card on file will be charged\*

## Review Charges

 Review Charges and Authorize

Please review the charges below and click authorize to begin your case. You will not be billed until your case is complete.

<p>3D Conversion &amp; Auto Approval Treatment Services <span style="float: right;">\$70</span></p>	<p>Subtotal: <span style="float: right;">\$295</span></p>
<p>Implant Concierge Surgical Guide In2Guide/YouFit Guided Kit - Maxilla - 1 Implant(s) <span style="float: right;">\$225</span></p>	<p>Patient Total: <span style="float: right;">\$0</span></p> <p>Other: <span style="float: right;">\$0</span></p>

Do you have a voucher code?

If you have received a voucher you may enter the code below to redeem it on this case.

Promo Code:  + Apply

[Where do I get one?](#)

**Your Total: \$295**

✔ Authorize

Update my billing info

Your credit card will not be billed until your Implant Concierge™ case is complete.