

1



Create case at www.implantconcierge.com

Start case by selecting "Fully Edentulous" button to extract all teeth. Complete Ideal Restorative Plan tooth chart. **Implant Concierge™** will provide a customized patient specific workflow.

2



Upload CBCT Scan #1 Denture Only

Apply 6 x-ray markers to a well-fitting, hard relined denture. Place denture on foam block and capture entire denture.

FREE

3



Upload CBCT Scan #2 Patient & Denture

Capture patient wearing denture with x-ray markers and a bite registration.

4



Schedule Virtual Implant Plan

Finalize your ideal restorative based treatment plan with an experienced 3D Case Coordinator in minutes. Schedule your VIP meeting online at your convenience!

No software necessary!

\$150

5



CAD/CAM Surgical Guide

Save surgical chair-time while achieving minimally invasive and precise implant placement. Compatible with most all implant brands.

Six implants, one implant, pilot, fully guided, tooth borne or tissue born **ALWAYS \$225!**

\$225