

1. Login to Implant Concierge. Click on “Create Implant Concierge Case”



2. Specify the patient information, guided surgical kit, and implant system in the corresponding fields.

Patient Information

First Name

Last Name

Date of Birth

Gender

Surgery Information

Surgery Date

If no date has been set, please provide an expected date so that we may prioritize your cases.

Implant Information

Guided Kit  [What is this?](#)

Implant Manufacturer

Brand

Choose Arch

- Maxilla Only
- Mandible Only
- Maxilla and Mandible

Next >

Then click the blue “Next” button.

3. Fill out the tooth chart on *Step 1* to reflect the patient's dentition at the time of the scan. Click on a tooth once to designate it as "missing". Click on the tooth a second time to show a dotted outline of the tooth, indicating it will be extracted at the time of surgery. Clicking the tooth a third time places the tooth back on the chart.

Create Treatment Plan

Fully Edentulous


Reset Charts

### Step 1



Click on teeth to mark as "Missing" or "To Be Extracted" at time of surgery

### Dentate Status At Time Of Scan

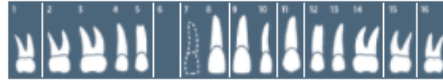

4. For the tooth chart on *Step 2 (Proposed Restorative Plan)*, please specify your *ideal* implant sites. Click on the pontic site once to have a virtual implant placed there.

### Step 2



Click on pontic icon to indicate proposed implant location

### Proposed Restorative Plan




5. **CBCT Scan:** Specify how the CBCT was captured and click the blue “Next” button. Please note, CBCT will be uploaded after the case has been created.

**CBCT Scan**

- Radiologist Interpretation
- Treatment Planning
- Surgical Guide - Maxilla
- Restorations

### CBCT Scan




**My CBCT**

You have your own CBCT machine in your office. ...

[More Info...](#)

**+ Select**




**My Scan Partner**

I will get my CBCT scan from a local imaging center or colleague....

[More Info...](#)

**+ Select**



**IC Scan Partner**

Have a local IC Scan partner acquire a CBCT scan for your patient....

[More Info...](#)

**+ 240.00**

**Next** >

6. **Radiologist Interpretation:** If you have any anatomical or pathological concerns, you can have your CBCT interpreted by our in-house Board Certified Maxillo-Facial Radiologist. Please provide your area of interest or instructions to the radiologist in the provided text box. Please note, this is an optional service. If no interpretation is needed, please select “No Radiology Interpretation” and click the blue “Next” button.

CBCT Scan


**Radiologist Interpretation**

Treatment Planning

Surgical Guide - Maxilla

Restorations


### Radiologist Interpretation

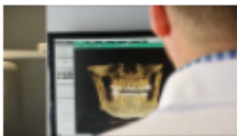

#### Radiology Interpretation

If you have pathological concerns or notice abnormal anatomy on your patient's CBCT, have your sca...

[More Info...](#)

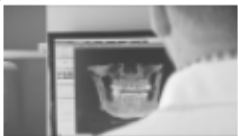

#### No Radiology Interpretation

I do not need a Radiology Interpretation for this case. ...

[More Info...](#)

## 7. Treatment Planning:

**Auto-Approval** -This is a streamlined option for single implant cases! Implant Concierge will still act as your virtual employee and create an initial treatment plan from your CBCT and STL files. However, in place of a meeting, we will create a 2-3-minute patient-specific video and will also include screenshots of different angles and measurements for your review. Select Auto-Approval and click the blue “Next” button. Please note, your charges will reflect on the left-hand side.

**\*Auto-approval services are ONLY available for SINGLE IMPLANT cases\***

The screenshot displays a web interface for 'Treatment Planning'. On the left is a vertical menu with options: 'CBCT Scan', 'Radiologist Interpretation', 'Treatment Planning' (highlighted), 'Surgical Guide - Maxilla', and 'Restorations'. The main content area is titled 'Treatment Planning' and features a video player icon. Below the icon, the text reads: 'Auto-Approval', 'A streamlined option for single implant cases! Implant Concierge will still act as your virtual emp...', and a 'More Info...' link. A 'Remove' button is at the bottom of this section. On the right, a 'Charges' table shows 'Auto-Approval' for '\$70.00' and a total of '\$70'. At the bottom of the interface are three buttons: 'Back', 'Next', and 'Complete Order'.

Charges	
Auto-Approval	\$70.00
	\$70

**8. Surgical Guide:** Select your tooth borne guide and click the blue “Next” button. Please note, your charges should be updated to include the surgical guide.

CBCT Scan


Radiologist Interpretation

Treatment Planning

Surgical Guide - Maxilla

Restorations

### Surgical Guide - Maxilla



#### Tooth Borne

The most predictable and stable option! The surgical guide will rest directly on the patient's exi...

[More Info...](#)

✕ Remove

#### Charges

Virtual Implant Plan	\$150.00
Tooth Borne	\$225
	<b>\$375</b>

← Back
Next →
✓ Complete Order

**9. Restorations:** Please utilize the tooth chart if you are planning on requesting restorative services and products. If you would like to order a

wax-up, please click on each tooth you would like included in the wax up and it will be reflected with a pink outline.

If you are planning on ordering a provisional, please click on a tooth twice to specify the requested provisional units Please note that the provisional charge will include a wax-up from Implant Concierge as well.

Once the tooth chart has been filled out, please fill out the restorative fields below. If no restoration services are requested, you can skip this section completely and click the green “Complete Order” button.

CBCT Scan

Radiologist Interpretation


Treatment Planning

Surgical Guide - Maxilla

**Restorations**

### Restorative

To order a virtual waxup or surgery-ready provisional, click on restorative tooth chart.

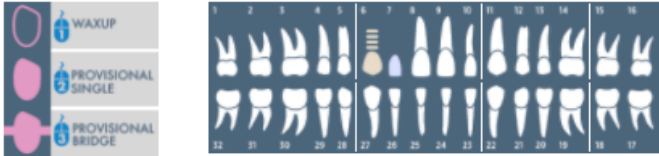

The restorative section features a vertical menu on the left with three options: 'WAXUP' (1), 'PROVISIONAL SINGLE' (2), and 'PROVISIONAL BRIDGE' (3). To the right is a dental tooth chart with 16 numbered positions (1-16) for upper teeth and 16 numbered positions (17-32) for lower teeth. A pink outline highlights tooth 11 on the upper chart.

Select Provisional Shade

Select an option

Proposed Restorative Plan

Select an option

Ideal Surgical Implant Protocol

Select an option

Ideal Prosthetic Plan

Select an option

3D Printed Model + \$50.00

Essix with Teeth + \$100.00

### Charges

Virtual Implant Plan	\$150.00
Tooth Borne	\$225
	<b>\$375</b>

◀ Back

✔ Complete Order

10. A summary of the requested services and products will now be shown. This page is for you to authorize our Implant Concierge team to begin working on your case. No charges will be incurred until the case is completed, therefore requested services and products can be adjusted throughout the life of the case as needed.

## Review Charges

### Review Charges and Authorize

Please review the charges below and click authorize to begin your case. You will not be billed until your case is complete.

Virtual Implant Plan		Subtotal:	\$375
Virtual Implant Plan	\$150	Patient Total:	\$0
<hr/>		Other:	\$0
Implant Concierge Surgical Guide - Tooth Borne		<b>Your Total: \$375</b>	
-- Maxilla - 1 implant(s)	\$225		

 **Authorize**

You will not be billed until your Implant Concierge™ case is complete.