



# Export Invivo/Anatomage Treatment Plan

In addition to the Invivo/Anatomage Treatment plan, Implant Concierge also requires the raw DICOM files, as well as a diagnostic cast (a digital impression or a stone model). More information can be found at the bottom of the page.

1. On your desktop, right click on an empty space. Go to “New” and click “Folder”. A folder will appear with highlighted blue text. Type in the patient name and push “Enter” on your keyboard.

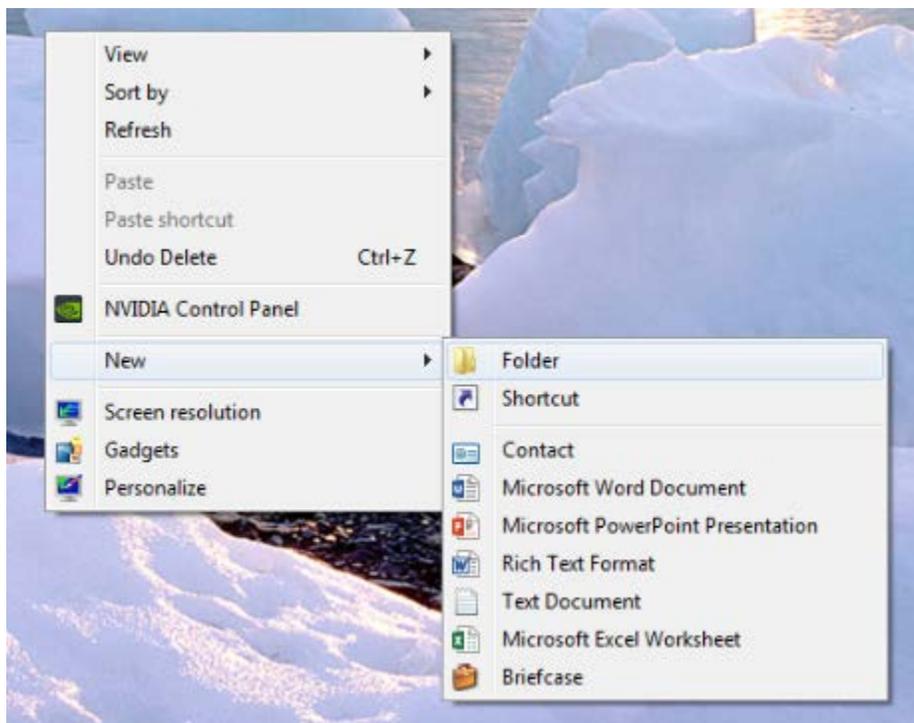


2. Once you have completed your treatment plan, go to “File”, and click “Save As”.

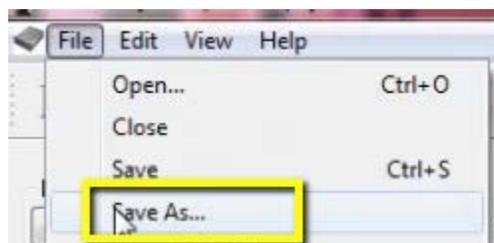


3. Select the option “Invivo” and click “OK”.




4. Select the folder you just created on the desktop and click “OK”.

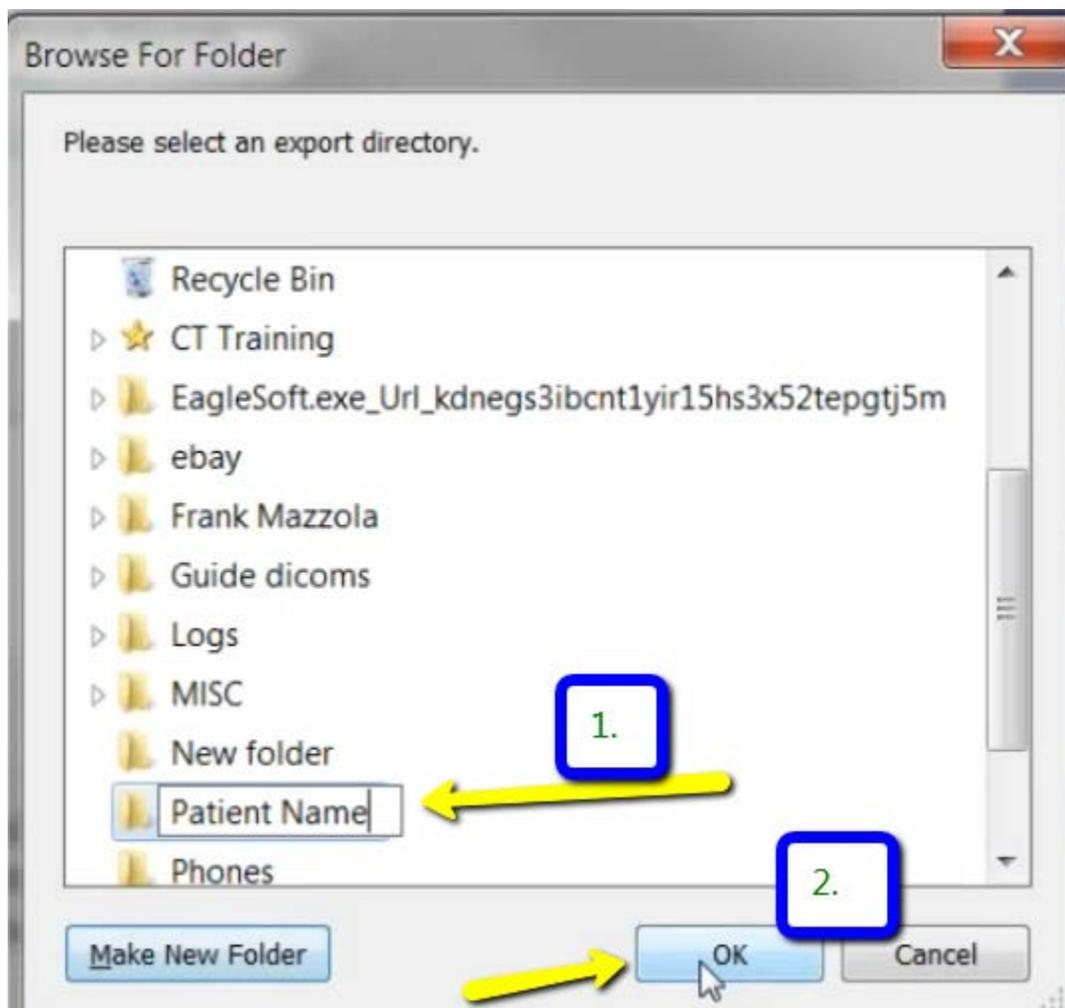

5. A final pop-up will appear. Select “Full” and click “Save”.

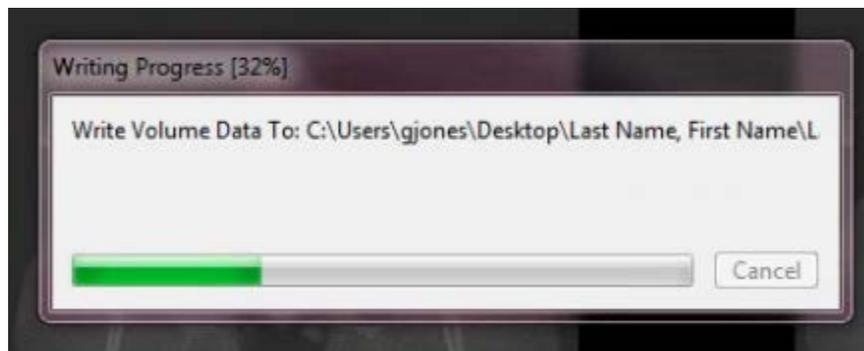

6. Go back to the desktop, right click on the patients folder, go to “Send to”, and select “Compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “Z”.

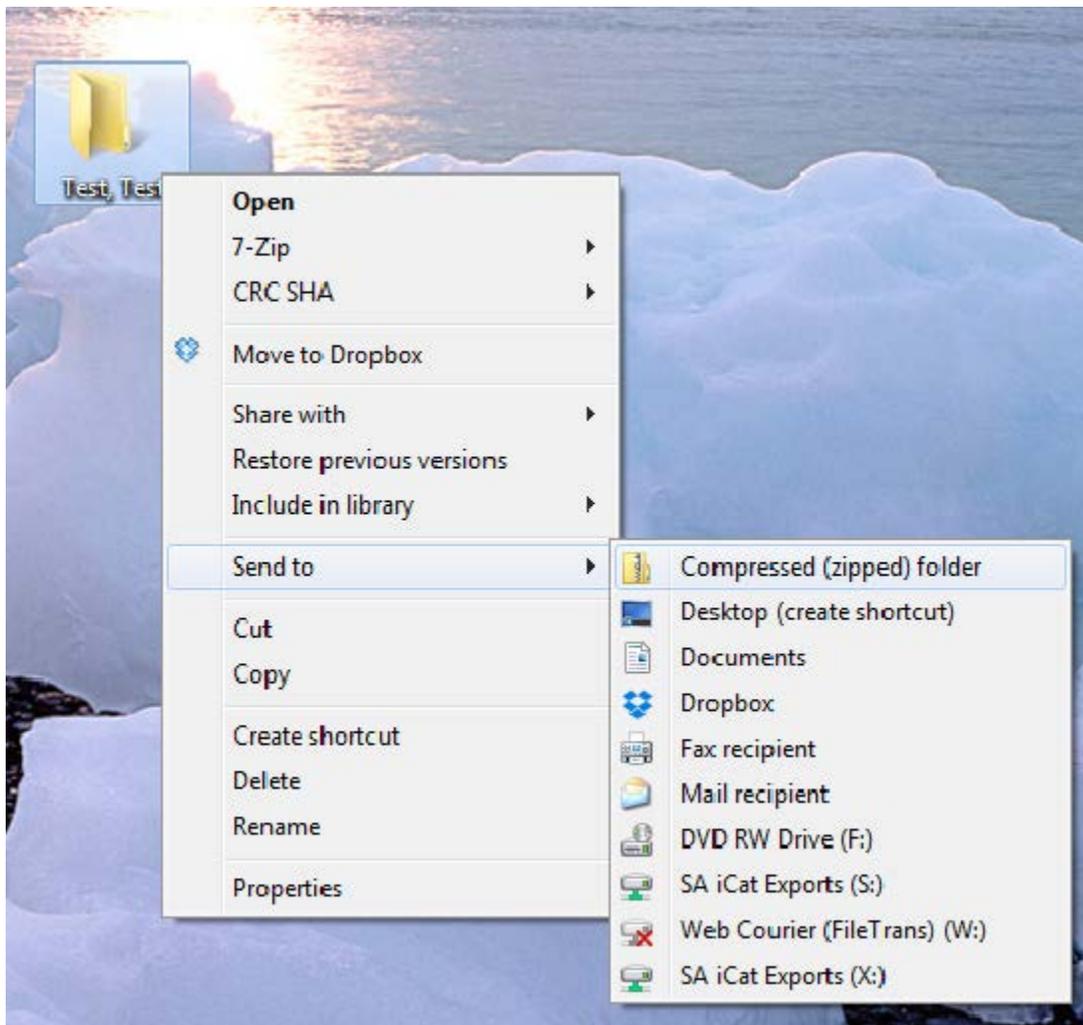

7. To upload the compressed folder, login to Implant Concierge. Click on the patient's name to open the case. Click on the red "Upload DICOM" button.

implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

## Current Cases

In Progress Pending Partner Completed All Cases

Invoice(s)

⌚ Indicates an action that has been delayed.

**Test, Test**

Dr. Demo  
Status: New Case  
Created: Jun 28, 2017

- ✔ Status: Implant Concierge Treatment Planning Case Created
- ⌚ Pending: Dr. Demo to upload DICOM Multi Files
- ⌚ Pending: Dr. Demo to upload .STL, or ship Diagnostic Model to Implant Concierge

implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

Case #1380740920 has been created and is waiting for its first action to be completed.  
Need help getting started? Call Customer Support: 866-977-2228

Dr. Test Demo

Case Files

+ Additional File(s)

### Test Test

Gender: Male | DOB: 0000

Options Case ID: 1380740920

Next Step

Dr. Demo to upload DICOM Multi Files

Upload DICOM Files



8. Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.



Upload DICOM Files

Patient: Test Test  
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files  
or use the Add DICOM File button below


Once all files have been uploaded click Complete

+ Add DICOM File

Cancel Complete

## Upload DICOM Files

Patient: Test Test  
Case Number: 1380740920

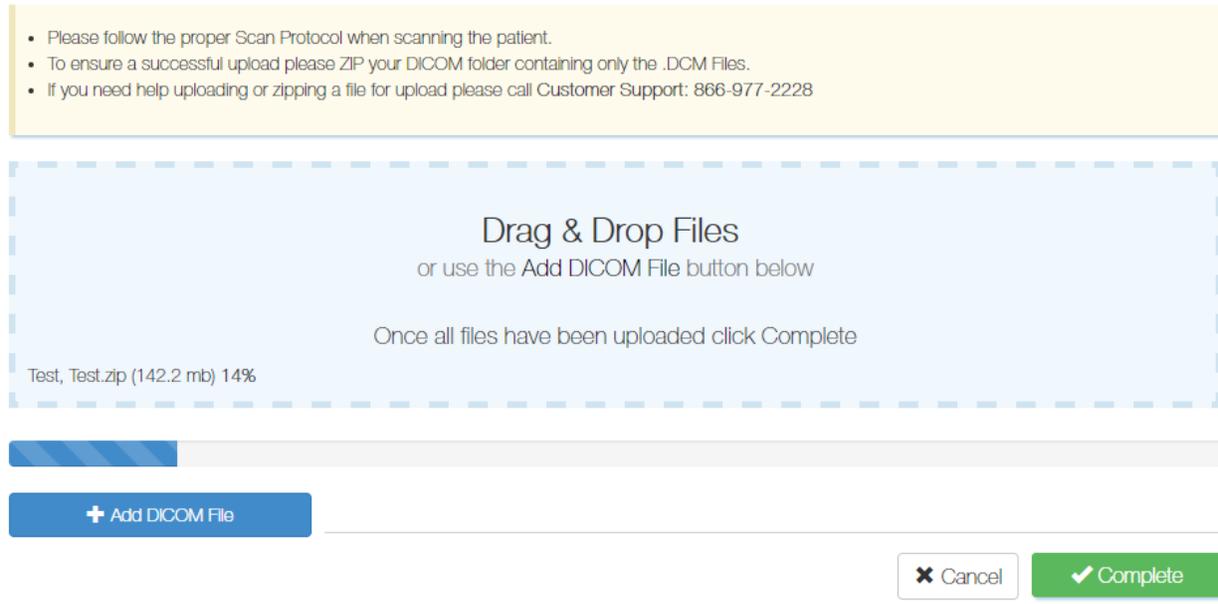

Upload DICOM Files

Patient: Test Test  
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files  
or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 14%

+ Add DICOM File

Cancel Complete

9. Once the file reaches 100%, click the green “Complete” button.

## Upload DICOM Files

Patient: Test Test

Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files  
or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 100%

+ Add DICOM File

✕ Cancel

✓ Complete

For more information regarding Exporting DICOM and sending diagnostic casts, please use the links below:

[Exporting DICOM from TxSTUDIO/Invivo](#)

[How To Print a Shipping Label](#)