



Exporting from VaTech (Ez3D-i)


1. On your desktop, right click on an empty space. Go to “New” and click “Folder”. A folder will appear with highlighted blue text. Type in the patient name and push “Enter” on your keyboard.

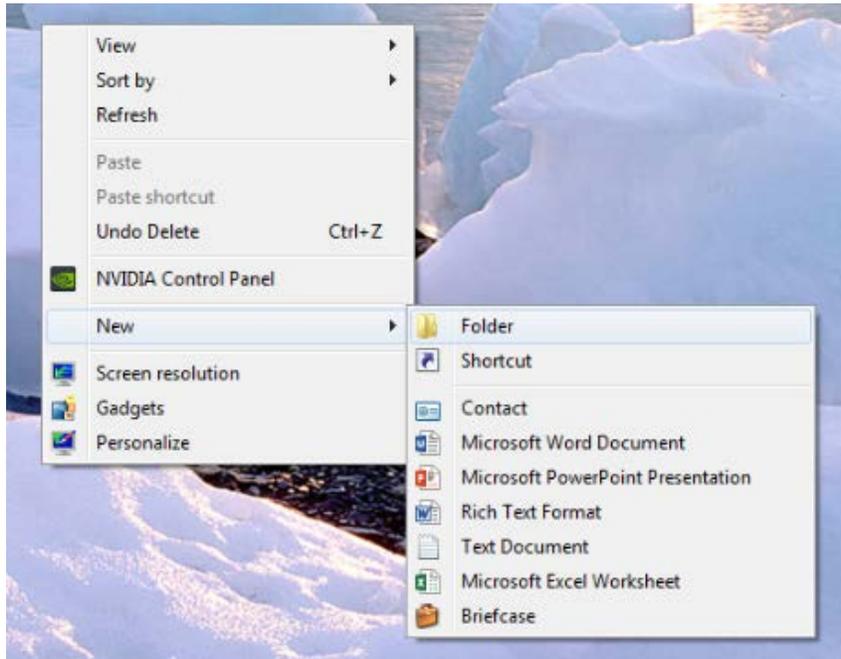




2. In your VaTech, open the patient scan. Click on “Main Menu”, then “Export”.

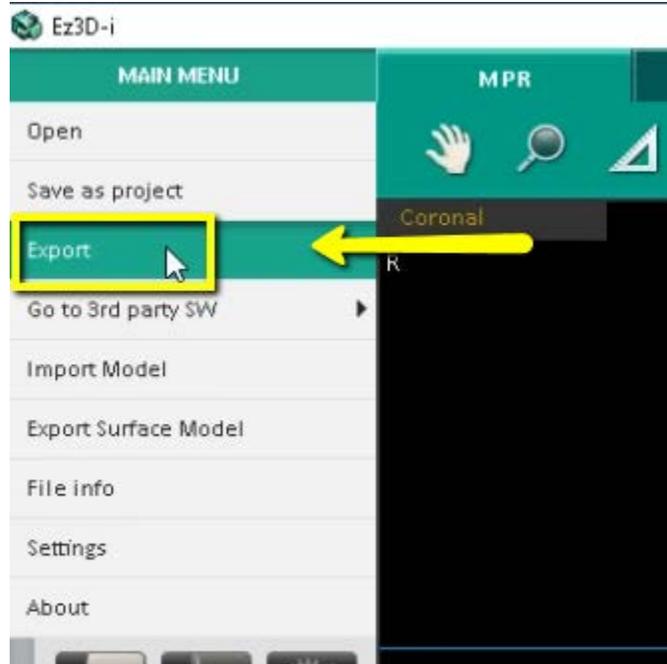


3. A pop up box will appear. For “Save Type”, select “Local Disk”. For Compression, select “Lossless”. Then check the items that you would like to export. Implant Concierge requires only the “DCM Files”. However, if you are a scanning site and would like to send the viewer to the Doctor, select “Viewer”. If you are wanting to export your treatment plan, select “Project File”. Then, click “Save”.

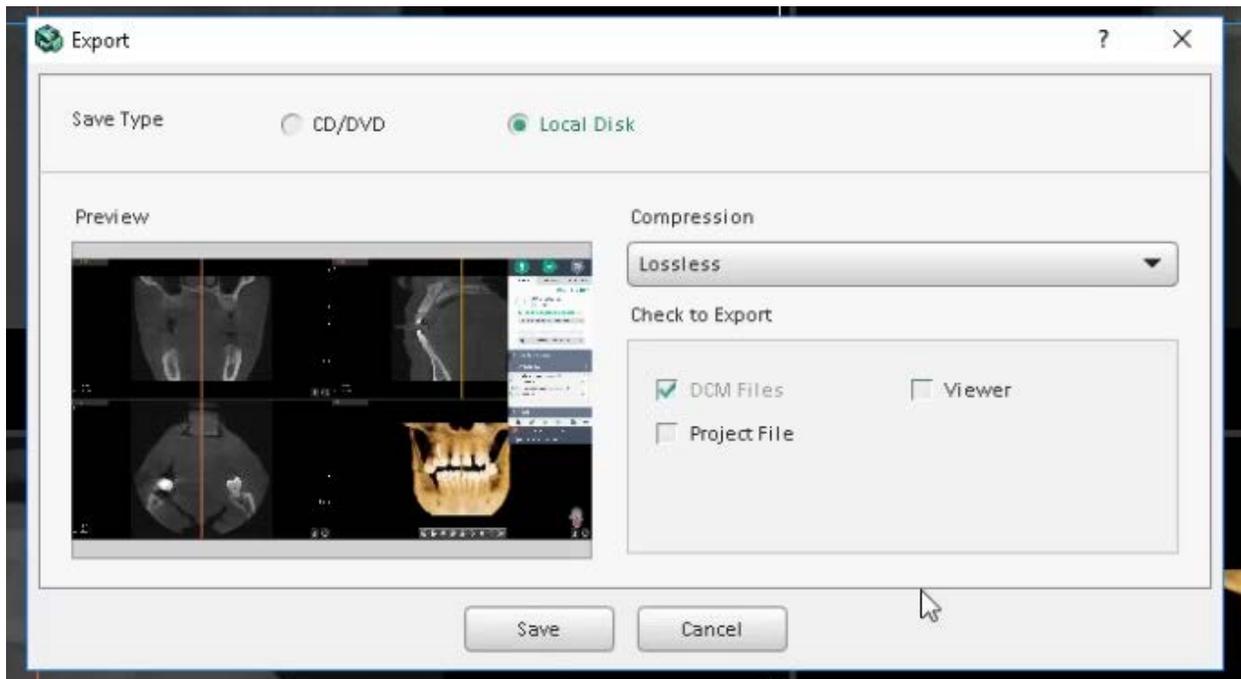

4. A second pop-up will appear for the save location. Select the folder you just created on the desktop, and click “Select Folder”. The DICOM files will begin to export. Once the files are exported, you will get a confirmation pop-up. Click “OK”.

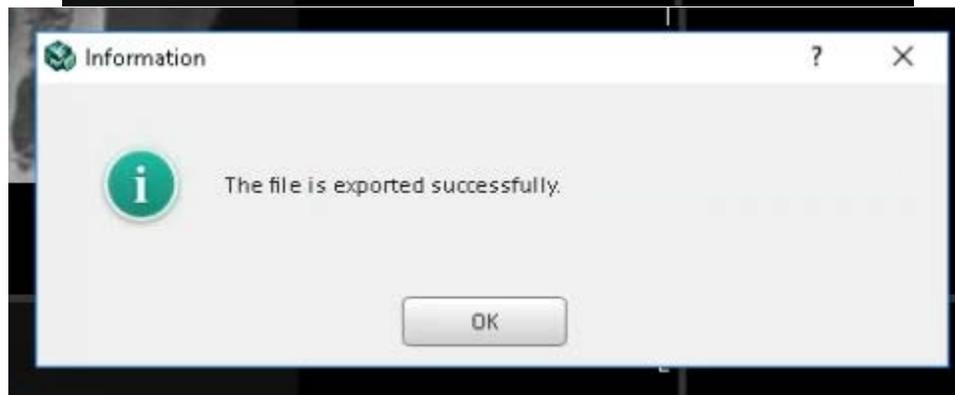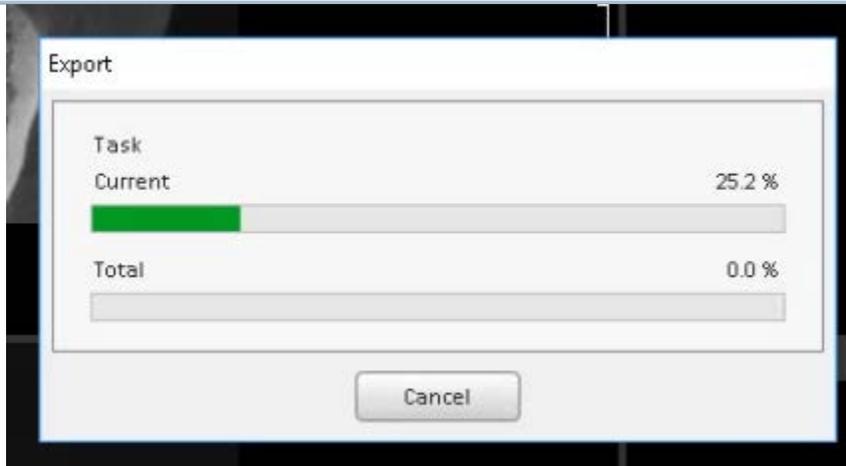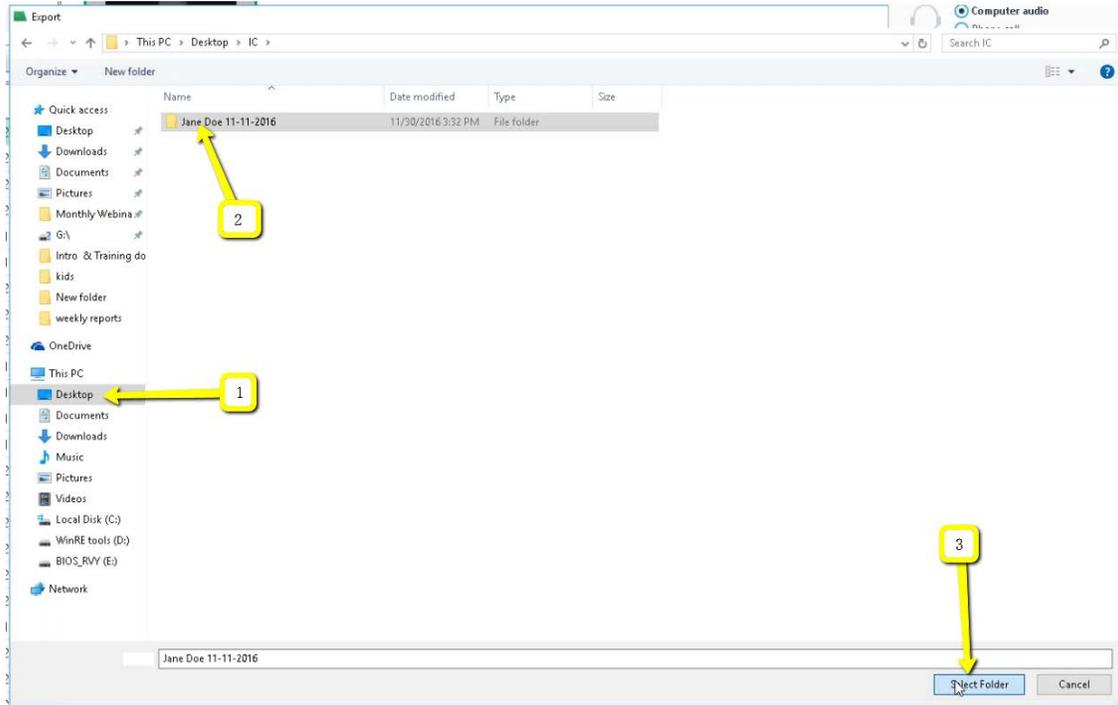

5. After the DICOM exports, go to the patients folder and verify the .dcm files were exported correctly; there should be several hundred files.

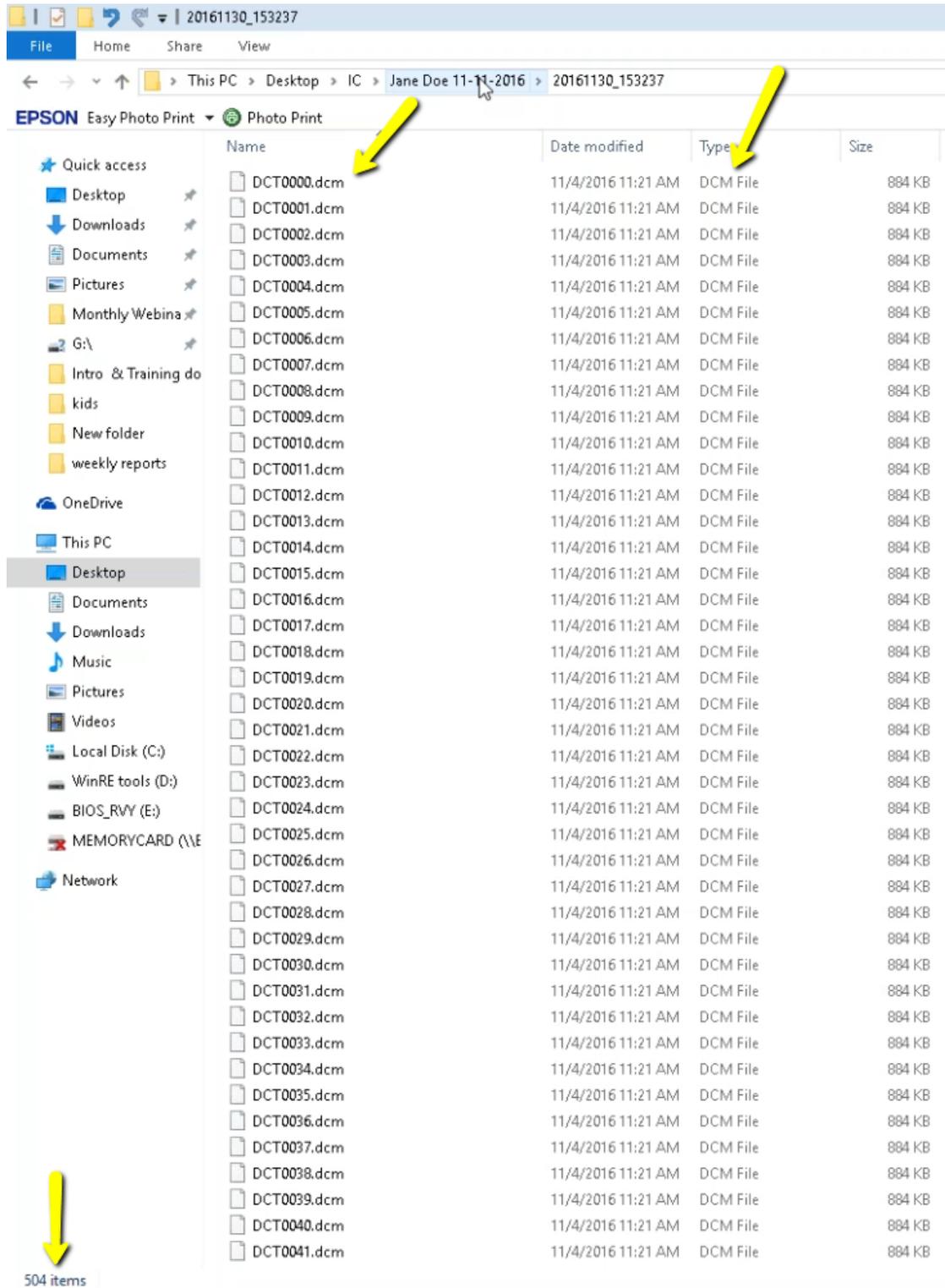


The screenshot shows a Windows File Explorer window with the address bar path: This PC > Desktop > IC > Jane Doe 11-11-2016 > 20161130_153237. The main pane displays a list of files with columns for Name, Date modified, Type, and Size. The files are named DCT0000.dcm through DCT0041.dcm, all with a date modified of 11/4/2016 11:21 AM and a size of 884 KB. A yellow arrow points to the 'Name' column header, another to the 'Type' column header, and a third to the 'DCT0000.dcm' file. A fourth yellow arrow points to the '504 items' text at the bottom left of the window.

Name	Date modified	Type	Size
DCT0000.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0001.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0002.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0003.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0004.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0005.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0006.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0007.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0008.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0009.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0010.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0011.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0012.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0013.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0014.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0015.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0016.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0017.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0018.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0019.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0020.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0021.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0022.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0023.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0024.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0025.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0026.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0027.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0028.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0029.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0030.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0031.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0032.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0033.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0034.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0035.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0036.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0037.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0038.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0039.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0040.dcm	11/4/2016 11:21 AM	DCM File	884 KB
DCT0041.dcm	11/4/2016 11:21 AM	DCM File	884 KB

504 items

6. Go back to the desktop, right click on the patients folder, go to “Send to”, and select “Compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “Z”.

(*** If following a Dual Scan Protocol, please zip scans individually and label accordingly; i.e: “Patient Scan” and “Denture Scan” ***)

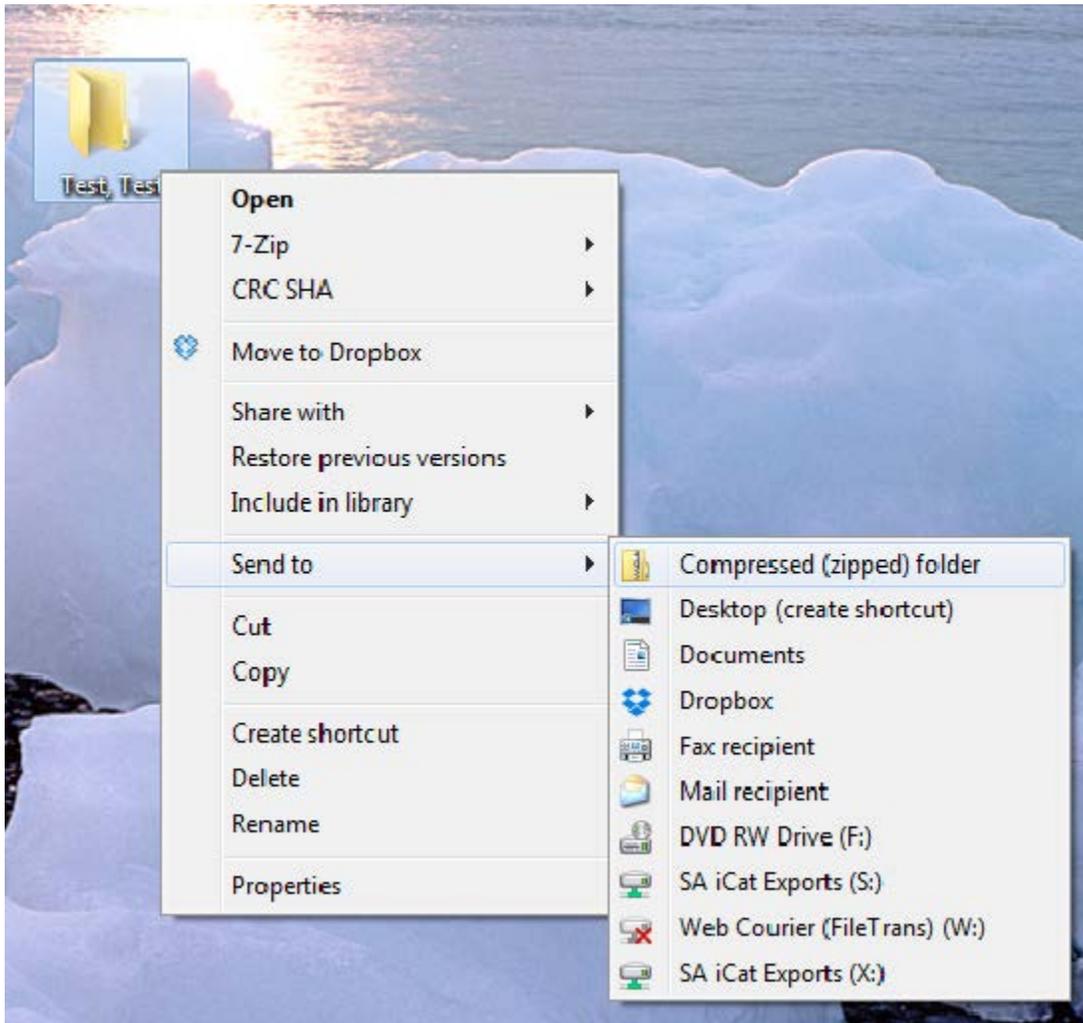

7. To upload the compressed folder, login to Implant Concierge. Click on the patient's name to open the case. Click on the red "Upload DICOM" button.



implant concierge Search for Patient Need Help? Settings Logout


View Current Cases Create Implant Concierge Case Order Individual Services

Current Cases

In Progress Pending Partner Completed All Cases

Invoice(s)

⏸ Indicates an action that has been delayed.

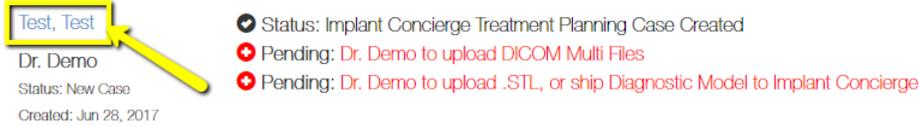

Test, Test
Dr. Demo
Status: New Case
Created: Jun 28, 2017

- ✔ Status: Implant Concierge Treatment Planning Case Created
- ⏸ Pending: Dr. Demo to upload DICOM Multi Files
- ⏸ Pending: Dr. Demo to upload .STL, or ship Diagnostic Model to Implant Concierge



implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

Case #1380740920 has been created and is waiting for its first action to be completed.
Need help getting started? Call Customer Support: 866-977-2228

Dr. Test Demo

Test Test

Options

Gender: Male | DOB: 0000

Case ID: 1380740920

Case Files


+ Additional File(s)

Next Step

Dr. Demo to upload DICOM Multi Files

Upload DICOM Files

8. Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.

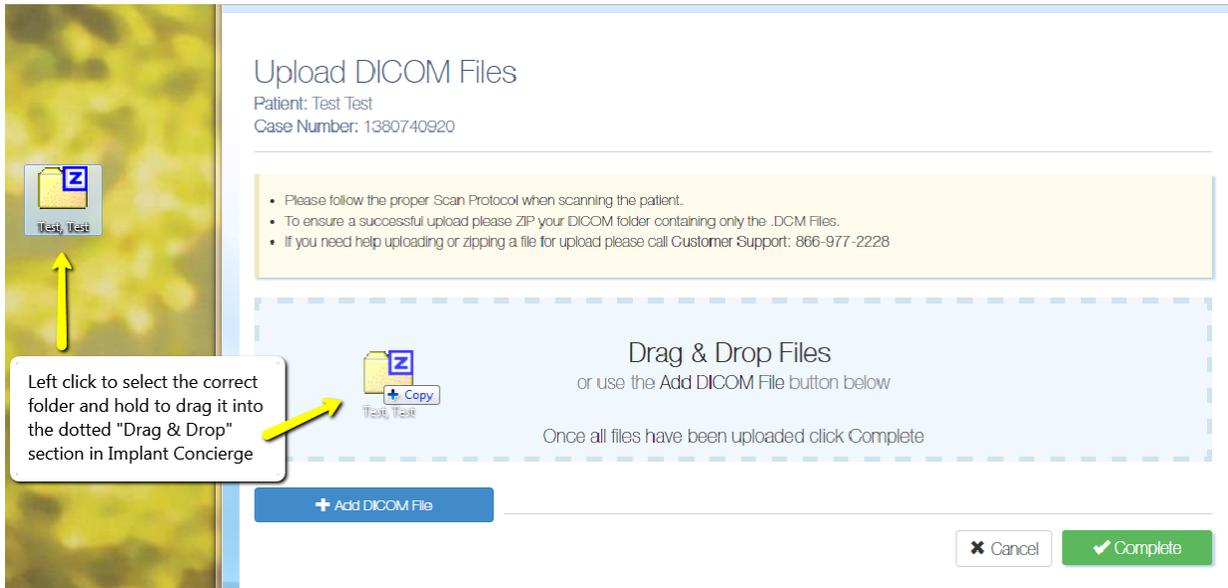

Upload DICOM Files

Patient: Test Test
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files
or use the Add DICOM File button below

Once all files have been uploaded click Complete

+ Add DICOM File

Cancel Complete

Upload DICOM Files

Patient: Test Test
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files

or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 14%

+ Add DICOM File

Cancel

Complete

9. Once the file reaches 100%, click the green “Complete” button.

Upload DICOM Files

Patient: Test Test

Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files

or use the Add DICOM File button below

Once all files have been uploaded click Complete



Test, Test.zip (142.2 mb) 100%

+ Add DICOM File

✕ Cancel

✓ Complete