

Creating a Partially Edentulous Case - Implant Concierge + You

1. Login to Implant Concierge. Click on “Create Implant Concierge Case”



2. Specify ALL fields on this page. If a surgery date has not been scheduled, please put a date you would like the surgical guide by.

Patient Information

First Name

Last Name

Date of Birth

Gender

Surgery Information

Surgery Date

If no date has been set, please provide an expected date so that we may prioritize your cases.

Implant Information

Guided Kit [What is this?](#)

Implant Manufacturer

Brand

Choose Arch

- Maxilla Only
- Mandible Only
- Maxilla and Mandible

Next >

Then click the blue “Next” button.

Creating a Partially Edentulous Case - Implant Concierge + You

3. Fill out the tooth chart on *Step 1* to reflect the patient's dentition at the time of the scan. Click on a tooth once to designate it as "missing". Click on the tooth twice to show a dotted outline of the tooth, indicating it will be extracted at the time of surgery. Clicking the tooth, a third time places the tooth back on the chart as present.

Create Treatment Plan

Fully Edentulous

Reset Charts

Step 1



Click on teeth to mark as "Missing" or "To Be Extracted" at time of surgery

Dentate Status At Time Of Scan




4. For the tooth chart on *Step 2 (Proposed Restorative Plan)*, please specify your *ideal* implant sites. Click on the pontic site once to have a virtual implant placed there.

Step 2



Click on pontic icon to indicate proposed implant location

Proposed Restorative Plan

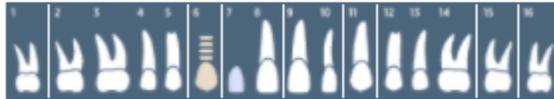

Creating a Partially Edentulous Case - Implant Concierge + You

5. **CBCT Scan: Specify how the CBCT was captured and click the blue “Next” button. Please note, CBCT will be uploaded after the case has been created.**

CBCT Scan

- Radiologist Interpretation
- Treatment Planning
- Planning Preferences
- Surgical Guide - Maxilla
- Additional Options
- Restorations

CBCT Scan



My CBCT

You have your own CBCT machine in your office. ...

[More Info...](#)

+ Select



My Scan Partner

I will get my CBCT scan from a local imaging center or colleague....

[More Info...](#)

+ Select



IC Scan Partner

Have a local IC Scan partner acquire a CBCT scan for your patient....

[More Info...](#)

+ 240.00

Creating a Partially Edentulous Case - Implant Concierge + You

- Radiologist Interpretation:** If you have any anatomical or pathological concerns, you can have your CBCT interpreted by a Board Certified Maxillo-Facial Radiologist. Please provide your area of interest or specify a general reading in the provided text box. Please note, this is an optional service. If no interpretation is needed, please select “No Radiology Interpretation” and click the blue “Next” button.

CBCT Scan

Radiologist Interpretation

Treatment Planning

Planning Preferences

Surgical Guide - Maxilla

Additional Options

Restorations


Radiologist Interpretation



Radiology Interpretation

If you have pathological concerns or notice abnormal anatomy on your patient's CBCT, have your sca...

[More Info...](#)

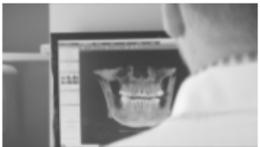

No Radiology Interpretation

I do not need a Radiology Interpretation for this case.

...

[More Info...](#)

Creating a Partially Edentulous Case - Implant Concierge + You

7. Treatment Planning: Implant Concierge + YOU

- You hire Implant Concierge to be your virtual employee to create your initial treatment plan. You, the licensed clinician, will choose how to finalize the treatment plan.

Choose one of the three Finalize Options:

1. I will finalize using RealGUIDE software
 - a. You will download the free RealGUIDE planning software, download the treatment plan that Implant Concierge created, and finalize the implant placement yourself.
2. Implant Concierge Auto-Approval
 - a. Implant Concierge will finalize the plan by providing you a recording and screenshots of the implant placement in the software.
 - b. Note Auto Approval services are for SINGLE implants only
3. Implant Virtual Implant Planning
 - a. Implant Concierge will host an online virtual meeting with an experience Case Coordinator and finalize the treatment plan with you.

CBCT Scan

Radiologist Interpretation

Treatment Planning

Planning Preferences

Surgical Guide - Maxilla

Additional Options

Restorations

Treatment Planning



Implant Concierge + You

Implant Concierge provides an initial treatment plan (DICOM conversion, model registration, implant ...)

[More Info...](#)

+ 75.00

Remove

Required: Finalize Treatment Plan Options

I will finalize using RealGUIDE Software

Implant Concierge Auto Approval - \$25.00

Implant Concierge VIP - \$50.00

Charges

Implant Concierge + You	\$75.00
	\$75

Creating a Partially Edentulous Case - Implant Concierge + You

8. Planning Preferences:

- a. Select your preferred Restorative Emergence, Ideal surgical implant protocol, and ideal prosthetic plan. Keep in mind, this is not required for all cases.

CBCT Scan	Planning Preferences
Radiologist Interpretation	
Treatment Planning	
Planning Preferences	
Surgical Guide - Maxilla	
Additional Options	
Restorations	
Preferred Restorative Emergence	
<input type="text" value="Select an option"/>	
Ideal Surgical Implant Protocol	
<input type="text" value="Select an option"/>	
Ideal Prosthetic Plan	
<input type="text" value="Select an option"/>	

Creating a Partially Edentulous Case - Implant Concierge + You

9. **Surgical Guide:** Select your tooth borne guide and click the blue “Next” button. Please note, your charges should be updated to include the surgical guide.

- CBCT Scan
- Radiologist Interpretation
- Treatment Planning
- Planning Preferences
- Surgical Guide - Maxilla**
- Additional Options
- Restorations

Surgical Guide - Maxilla



Tooth Borne

The most predictable and stable option! The surgical guide will rest directly on the patient's exi...

[More Info...](#)

+ 225.00

Remove

Charges

Implant Concierge + You	\$75.00
Tooth Borne	\$225.00
	\$300

Creating a Partially Edentulous Case - Implant Concierge + You

10. Additional Options

a. Select additional products to be added to your case, if needed.

CBCT Scan

Radiologist Interpretation

Treatment Planning

Planning Preferences

Surgical Guide - Maxilla

Additional Options

Restorations

Additional Options

 <h4>Cerec Model Conversion</h4> <p>Implant Concierge to convert DXD file to STL file...</p> <p>More Info...</p> <div style="text-align: center; margin-top: 20px;">+ 18.00</div>	 <h4>3D Model Print</h4> <p>...</p> <p>More Info...</p> <div style="text-align: center; margin-top: 20px;">+ 25.00</div>	 <h4>3D Analog Model Print (does not include analog)</h4> <p>...</p> <p>More Info...</p> <div style="text-align: center; margin-top: 20px;">+ 50.00</div>
 <h4>Anchor Pin Drill</h4> <p>1.5mm Implant Concierge Anchor Pin Drill (reusable)...</p> <p>More Info...</p>	 <h4>Pilot Drill</h4> <p>Implant Concierge Pilot Drill...</p> <p>More Info...</p>	 <h4>Anchor Pins Only</h4> <p>3 Implant Concierge Anchor Pins (reusable)...</p> <p>More Info...</p>

Creating a Partially Edentulous Case - Implant Concierge + You

11. Restorations: *Optional Service

If you would like to have a virtual wax-up created, please click on the desired tooth once, and it will reflect with a pink outline.


If you would like a surgery-ready provisional, please click on the tooth twice, and a pink pontic will reflect. Please note, a virtual wax-up is included in the provisional fee.

If no restoration services are needed, you can skip this section completely and click the green “Complete Order” button.

- CBCT Scan
- Radiologist Interpretation
- Treatment Planning
- Planning Preferences
- Surgical Guide - Maxilla
- Additional Options
- Restorations**

Restorative

To order a virtual waxup or surgery-ready provisional, click on restorative tooth chart.

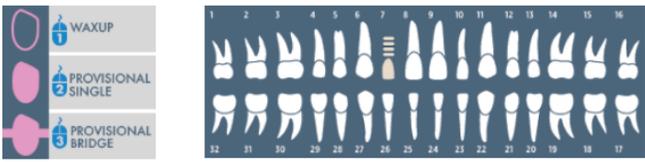

The image shows three options for restorative services: 1. WAXUP, 2. PROVISIONAL SINGLE, and 3. PROVISIONAL BRIDGE. To the right is a dental arch chart with teeth numbered 1 through 32. Teeth 1-16 are in the upper arch and 17-32 are in the lower arch. Tooth 26 is highlighted with a pink outline.

Select Provisional Shade

Select an option

Essix with Teeth + \$125.00

Charges

Implant Concierge + You	\$75.00
Tooth Borne	\$225.00
	\$300

Creating a Partially Edentulous Case - Implant Concierge + You

12. A summary of the requested services and products will be reflected on the Review Charges page. No charges will be incurred until the case is completed, therefore requested services and products can be adjusted throughout the life of the case as needed.

Review Charges

Review Charges and Authorize

Please review the charges below and click authorize to begin your case. You will not be billed until your case is complete.

Products & Services

Initial Treatment Plan (1 implant)

Implant Concierge to create initial treatment plan for single implant case

\$75

Implant Concierge Surgical Guide - Tooth Borne

BioHorizons - Tapered Internal - Maxilla - 1 implant(s)

\$225

Subtotal: \$300

Patient Total: \$0

Other: \$0

Your Total: \$300

 Authorize

You will not be billed until your Implant Concierge™ case is complete.